



Biocalis

BIOCALIS UROLOGY

- **DISPOGEL**® (Urethral lubricant gel with %2 lidocaine HCL).

www.biocalis.com

DISPOGEL[®]

(Urethral lubricant gel with %2 lidocaine HCL)



- Dispogel is a sterile lubricant gel containing local anesthetic (Lidocaine 2% and antiseptic (Chlorhexidine Digluconate 0.05%) Dispogel is available in both 6ml and 11ml single use pre-filled syringes each presented in a sterile package.
- Dispogel is used for instillation into the urethra prior to insertion of a catheter or other medical instruments. The insertion of urinary catheters together with Dispogel Urethral Lubricant Gel with Lidocaine HCL (11ml & 6ml) health professionals minimise associated trauma, pain and catheter associated infections.

DISPOGEL®

- DISPOGEL Urethral Lubricant Gel with %2 Lidocaine HCL is used to lubricate the urethral region prior to administration of the medical device to ensure the comfort of the patient during catheter, endoscopy and cystoscopy and other medical devices during the application of the urethral region.
- When the medical device is applied to the urethral region, the medical device is easily advance on a slippery surface. While the urethral gel is lubricated, it also provides the patient's comfort by temporarily numbing the urethral region with lidocaine to prevent pain.
- In addition, through the chlorhexidine contained in an antiseptic area, the patient is left in a safe and comfortable condition after use. Antiseptics reduces the risk of infection getting in, while lubricating to make the process smooth.

Dispogel Uretral Lubricant gel with 2 % Lidocaine HCL

Components	% g
Lidocaine Hydrochloride	2
Chlorhexidine Digluconate	0,05
Methyl Hydroxybenzoate	0,06
Propyl Hydroxybenzoate	0,02
Propylene Glycol	20
Hydroxy Ethyle cellulose	0,80
Purified Water	Up to 100 ml

PRODUCT USE AND APPLICATION



1- The syringe is removed from its sterile package by tearing off the backing paper

2- Before removing the blue cap from the end of the syringe, free the plunger by gently pressing it.





Step 1



Step 2



Step 3



Step 4



Step 5



Step 6




Step 7



Step 8

3- Remove the cap. Insert the nozzle into the opening of the urethral area to be of a catheter or instruments the gel. The anaesthetic takes 5 minutes to work after the gel has been administered and the

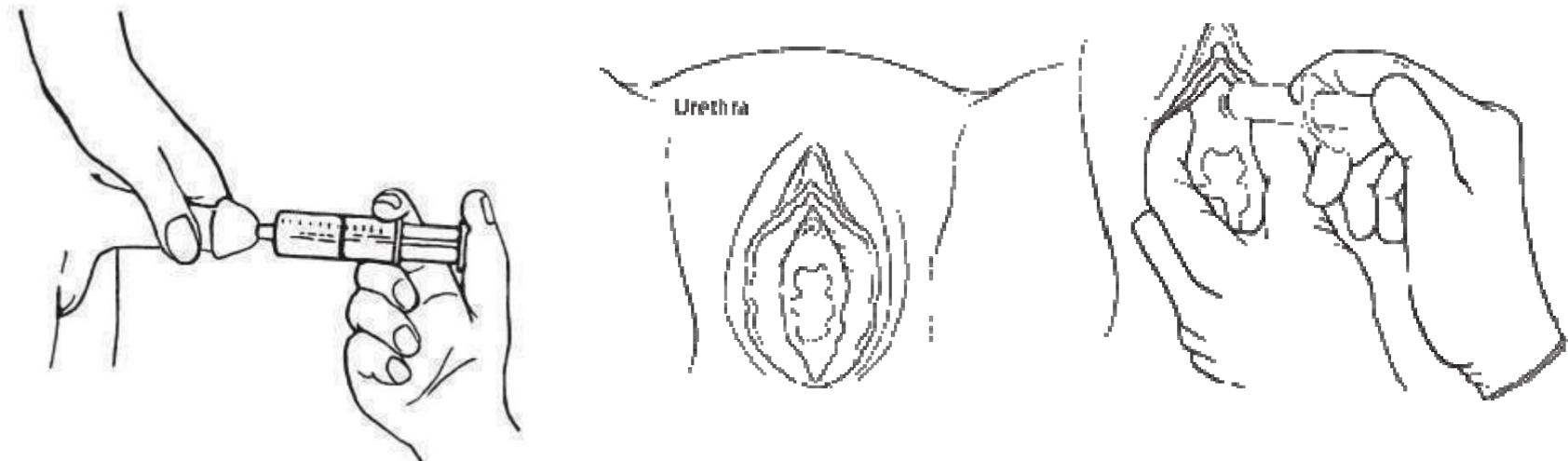

After instillation of gel, wait for 5-10 minutes prior to insertion of a catheter or other medical instruments – time needed for achieving anesthetic effect.

4- The product is disposed AS A medical waste after use.



Step 1



Step 2



Step 3



Step 4



Step 5

DISPOGEL®

(Urethral lubricant gel with %2 Lidocaine HCL)



6ml	11ml	STERILE R	CE 2292
GMDN Code	Product Description	Brand	Pack Size
37717	6 ml Syringe	Dispogel®	Pack of 50
37717	11 ml Syringe	Dispogel®	Pack of 25



Contact Us

Biocalis Ltd. Commercial Head Office

Tel (TR): +90 212 367 92 79 Fax: +90 212 366 58 02

Tel (UK): +44 20 8123 2633

Email : info@biocalis.com

**Adress: Merkez Mah., Kağıthane Cad., İ Ofis,
No:3/30, Kağıthane, İstanbul**



“Health is
our greatest wealth”

‘To serve as the most trusted, convenient multichannel provider of innovative health solutions, products and services by enabling access to high quality medical devices.’